

URINE TESTING ALIQUOT REQUIREMENTS

<p style="text-align: center;">URTS</p>  <p style="text-align: center;">Aliquot Transfer Tube Lawson order number 004411</p>	<p style="text-align: center;">URCP</p>  <p style="text-align: center;">Mother Cup Lawson order number 035712</p>	<p style="text-align: center;">URCN</p>  <p style="text-align: center;">Urine Conical Lawson order number Conical 063240 Caps 061974</p>	<p style="text-align: center;">UR60</p>  <p style="text-align: center;">Referrals Order through Client Supply</p>
<p>BKVUR MABSP COTN TP24 CRCLR TPURI/TPUR/PEPU CRE24 TPCUR CRUR TXPRO/TXSCR WORKSTATION RLC01 (LABCORP)</p>	<p>URCUL NONGY</p>	<p>PEPU (10ml MIN) URSCR UAPR URSED HCGU</p>	<p>WORKSTATION RLC26 (MAYO) WORKSTATION RLC49 (MISC)</p>
<p style="text-align: center;">VY</p>  <p style="text-align: center;">Viral Yellow</p>	<p style="text-align: center;">UR60-LP</p>  <p style="text-align: center;">Referrals – light protect</p>	<p style="text-align: center;">URTM</p>  <p style="text-align: center;">RLC30 – Med Tox</p>	<p style="text-align: center;">URG</p>  <p style="text-align: center;">RLC70– Green Top</p>
<p>CT/GC VAGPN Order through Client Supply</p>	<p>WORKSTATION RLC26/61 (MAYO) when light protection is called for Order through Client Supply</p>	<p>WORKSTATION RLC30 (MEDTOX) WORKSTATION RLC01 (LABCORP) Order through Client Supply</p>	<p>WORKSTATION RLC70 (MCL) (All toxicology quant levels) Order through Client Supply</p>

*TXPRO/TXSCR need 1 each of the white aliquot (URTS) and the Yellow Conical (URCN) (remember to only fill the URCN ¾ full)

*PEPU needs to be aliquoted into **BOTH** the white Aliquot (URTS) and the yellow Conical (URCN) – Either random or 24 hour is acceptable, minimum volume is 10mls.